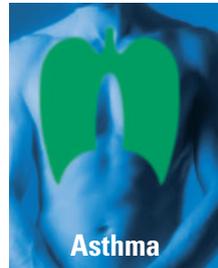
Hypertension

Human Fever

Asthma

Flexible Heating

EN Explanation of symbols

- ① Read the instructions carefully before using this device.
- ② Do not use folded or rucked.
- ③ Do not insert needles.
- ④ Not to be used by very young children (0 - 3 years).

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Ongain Electronic (Shenzhen) Co., Ltd., 16
Jiayi Industrial Area, Zhangqi Road, Guanlan
Street, Baoan District, Shenzhen, China

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IB.FH.all.EN.RU.2019

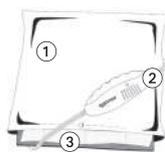
- **Destination:** the medical heating pad FH 80, FH 300 is for warming the human body.
- **Application scope:** This device is not intended for medical use in hospitals. For household use only.
- **To avoid any risk of electric shock when using the pad please observe the following:**
 - Do not use if wet. If the pad has become damp or wet, completely dry the pad before use.
 - Only use the pad in dry rooms, not e.g. in the bathroom.
 - Check that the pad is undamaged each time before use, i.e. do not use it if there is visible damage to the hand control, switches, plug or the pad itself.
 - Do not insert pins or other sharp objects.
- **To avoid overheating:**
 - Do not leave the pad switched on unsupervised; switch it off when you leave the room.
 - Do not use the pad while folded.
 - Avoid creasing the pad.
 - **Do not cover the hand control!** To ensure that the hand control functions correctly, do not place it on or under the heating pad or cover it with other objects.
 - The cable should not be wound around the hand control.
- **Do not use on a person incapable of movement, a baby, a person insensitive to heat, or animals.** Do not let children play with the device.
- **Ensure that children do not use this device unsupervised.** Children under 8 years of age must not use this device unless a parent or guardian has set the control to the lowest temperature setting 1.
- This device may be used by children aged 8 or over and by people who have reduced physical, sensory or mental abilities or inadequate experience and/or knowledge, provided they are supervised or have been instructed on how to use the device safely and have understood the resulting dangers.
- **Excessive application at a high setting can result in skin burns.**
- **When the device is likely to be used for an extended period, i.e. by the user falling asleep,** controls should be adjusted to a setting recommended for continuous use.
- **Examine the device frequently for signs of wear or damage.** If there are signs of wear or damage or the device has been misused, please contact the manufacturer or authorised agent prior to any further use.
- **If the power cable is damaged, it must be replaced by the manufacturer, an authorised agent or similarly qualified persons in order to avoid any hazards.**
- **Do not attempt to open or repair the device yourself.** Special tools are required to re-assemble the device.
- **Allow the device to cool down before folding and storing it in a dry place.** Do not place heavy objects on it to prevent buckling.

⚠ Not suitable for children between 0-3 years! Children under 3 years of age are unable to respond to overheating.

1. Contraindications

This device emits very weak electrical and magnetic fields which may disturb the function of your cardiac pacemaker. Therefore, we recommend you to consult your physician and the manufacturer of your cardiac pacemaker before using this device.

2. How to Use a Microlife Heating Pad

- ① Soft fabric cover – removable and machine washable. Fastener.
- ② Hand control with slider for heat settings and ON indicator.
- ③ Pad

⚠ WARNING: Never use the pad without the cloth cover. Allow some air circulation between the pad and skin during use.

1. Put the cloth cover ① on a flat surface. Open the fastener.
2. Insert the pad ③ completely into the cover, fitting one side at a time. Make sure the cable and hand control ② remain outside the cover.
3. The pad will lie flat, filling the entire cover. Close fastener.
4. Plug into a 220-240 volt AC mains socket and set the hand control to the desired heat setting using the slider.
 - Select the temperature setting on the slider – this switches on the heat pad. An indicator will illuminate, showing that the appliance is connected to the power supply and is switched on.
 - Setting 1 is recommended for continuous use.
 - The pad is switched off by moving the slider to the «0» position. The indicator light will then turn off.
5. Apply the heating pad to a necessary part of the body.
6. **The hand control will automatically turn off the heater after approximately 90 minutes,** then the indicator light will flash until reset or turn off. To reset the timer and start a further heating period, move the slider to the off «0» position and then back to the desired heat setting.
7. Unplug when finished.

3. Cleaning Instructions

⚠ WARNING: Always unplug the power cable from the mains socket before undertaking any cleaning of the product.

1. The cover can be machine washed after removal from the pad according to the washing instructions given on the cover.
2. The pad must **not** be machine washed or dry-cleaned. The FH 80, FH 300 (PVC surface) can be cleaned by using a soft damp cloth.
3. Do not allow the hand control to get wet. Do not use any detergents or solvents as they may damage the device.
4. Never hold the hand control under water. Otherwise moisture may enter the hand control and cause damage.
5. Check the hand control for signs of damage. Dry the pad and its fabric cover completely before switching it on again.

👉 Cleaning and user maintenance shall not be made by children without supervision.

4. Explanation of Symbols

Wash at 40 °C!

Do not iron!

Do not wash!

Do not dry-clean!

Bleaching not permitted!

Tumble-dry!

Iron at low temperature!

Do not dry in the tumble dryer!

Read the instructions carefully before using this device.

Type BF applied part

Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

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 Manufacturer

Operating temperature restriction
5 - 40 °C

Storage temperature restriction
-5 - +40 °C

Storage

5. Guarantee

This product is guaranteed for **2 years** from the date of purchase. This guarantee covers the hand control, cable, and the pad itself.

This guarantee does not cover incorrect use, accidents or damage resulting from failure to follow instructions, or modifications to the device not carried out by the manufacturer.

The guarantee is only valid upon presentation of the guarantee card completed by the dealer.

6. Technical Specifications

Model: FH 80, FH300

Voltage: 220 - 240 V

Power: 100 W

Operating temperature: 5 - 40 °C

Storage temperature: -5 - +40 °C

Frequency: 50 Hz

Included accessories: Removable cover, heating pad with hand control, instruction manual, guarantee card, packaging

Technical alterations reserved.

7. www.microlife.com

Detailed user information about our products and about our services can be found at www.microlife.com.

Date of production marked on the pad:

Lot number: mm/yyyy.

Date of production: (month/year): mm/yyyy.

Disposal

For disposal of products, please contact the specialized organizations authorized to conduct disposals in accordance with the legislation of the Russian Federation.

Standards compliance:

Declaration of conformity

- **Destination:** the medical heating blanket FH 422 is for warming the human body.
- **Application scope:** This device is not intended for medical use in hospitals: For household use only.
- **Use the heating blanket as under blanket only:**
 - The device is only to be used with the original hand control FH 422.
 - **Do not use if wet.** Before use, damp or wet heating blankets must be dried thoroughly.
 - **Do not use if folded.**
 - **Do not use if strongly creased.**
- **Before each use of the heating blanket make sure that it is not damaged.** It should not be used in case of visible damage of the hand control, switches, plug or the heating blanket itself. Please contact the manufacturer or authorised agent prior to any further use.
- **Do not use on a person incapable of movement, a baby, a person insensitive to heat, or animals.** Do not let children play with the device.
- **Ensure that children do not use this device unsupervised.** Children under 8 years of age must not use this device unless a parent or guardian has set the control to the lowest temperature setting 1.
- This device may be used by children aged 8 or over and by people who have reduced physical, sensory or mental abilities or inadequate experience and/or knowledge, provided they are supervised or have been instructed on how to use the device safely and have understood the resulting dangers.
- **Do not insert pins or other sharp objects.**
- **Do not cover the hand control!** To ensure that the hand control functions correctly, do not place it on or under the heating blanket or cover it with other objects.
- **Make sure that the heating blanket and the cord does not get trapped when used on an adjustable bed.**
- The cable should not be wound around the hand control.
- **If the power cable is damaged, it must be replaced by the manufacturer, an authorised agent or similarly qualified persons in order to avoid any hazards.**
- **Do not attempt to open or repair the device yourself.** Special tools are required to re-assemble the device.
- **Allow the device to cool down before folding and storing it in a dry place.** Do not place heavy objects on it to prevent buckling.

 **Not suitable for children between 0-3 years!** Children under 3 years of age are unable to respond to overheating.

1. Contraindications

This device emits very weak electrical and magnetic fields which may disturb the function of your cardiac pacemaker. Therefore, we recommend you to consult your physician and the manufacturer of your cardiac pacemaker before using this device.

2. How to Use a Microlife Heating Blanket

1. Beginning from the foot end spread out the heating blanket flat onto the mattress.
2. After this, pull up the sheet over the bed as usual, so that the heating blanket comes between the mattress and the sheet.
3. **Make sure that the heating blanket is spread out flat without any folds.** Check this while making the bed again as the heating blanket may only be used if it is spread out evenly.
4. Now you can switch on the heating blanket using the hand control. Level 1 corresponds to minimal and level 3 to maximal heating. **For a comfortably warm bed:** approx. 30 minutes before going to bed, switch on the heating blanket to level 3 and cover it with a quilt. After this reduce the heat to levels 1-2: This is especially suited for the use during the entire night.
5. **After 12 hours of continuous use, the heating blanket switches off automatically,** however the indicator light will remain illuminated. To reset the timer and start a new

heating period, move the slider to the off «0» position and then back to the desired heat setting.

6. **Unplug the heating blanket when finished.**

3. Cleaning Instructions

 **WARNING:** Always unplug the power cable from the mains socket before undertaking any cleaning of the product.

1. Detach the plug coupling (and therefore the switch) before starting the wash cycle. Do not allow the hand control to get wet.
2. The heating blanket must not be dry-cleaned.
3. **The heating blanket may be machine washed according to the washing instructions given on the blanket label.** Wash the heating blanket at 40 °C using the gentle wash program for sensitive textiles and a fine-fabric detergent.
4. For laundering put the heating blanket into a laundry bag.
5. Do not spin-dry or use a tumble-dryer.
6. Dry the heating blanket carefully. In order to do so, place it on a clothesline or a rail. Do not attach using clothes pegs. **Never switch the device on for drying.** Do not connect to the power supply before the blanket is completely dry.

 Frequent laundering leads to premature ageing. **Wear and tear as a consequence of frequent or incorrect laundering is excluded from the guarantee.**

 Cleaning and user maintenance shall not be made by children without supervision.

4. Explanation of Symbols

- | | | | |
|---|--|---|---------------------------------|
|  | Wash at 40 °C! |  | Do not iron! |
|  | Do not wash! |  | Do not dry-clean! |
|  | Bleaching not permitted! |  | Tumble-dry! |
|  | Iron at low temperature! |  | Do not dry in the tumble dryer! |
|  | Read the instructions carefully before using this device. | | |
|  | Type BF applied part | | |
|  | Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste. | | |

 Class II equipment

 Lot number

 Manufacturer

 **Operating**
Operating temperature restriction
5 - 40 °C

 **Storage**
Storage temperature restriction
-5 - +40 °C

5. Guarantee

This product is guaranteed for **2 years** from the date of purchase. This guarantee covers the hand control, cable, and the heating blanket itself. This guarantee does not cover incorrect use, acci-

dents or damage resulting from failure to follow instructions, or modifications to the device not carried out by the manufacturer.

The guarantee is only valid upon presentation of the guarantee card completed by the dealer.

6. Technical Specifications

Model: FH 422

Voltage: 220 - 240 V

Power: 90 W

Operating

temperature: 5 - 40 °C

Storage

temperature: -5 - +40 °C

Frequency: 50 Hz

Included Hand control with slider for heat settings,

accessories: heating blanket, instruction manual, guarantee card, packaging

Technical alterations reserved.

7. www.microlife.com

Detailed user information about our products and about our services can be found at www.microlife.com.

Date of production marked on the pad:

Lot number: mm/yyyy.

Date of production: (month/year): mm/yyyy.

Disposal

For disposal of products, please contact the specialized organizations authorized to conduct disposals in accordance with the legislation of the Russian Federation.

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Microlife AG